
HEINE BETA100® / HEINE K100® Diagnostik-Otoskope

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0 · Fax +49(0)81 52/38-202
E-mail: info@heine.com · www.heine.com
med 3898 1/9.07

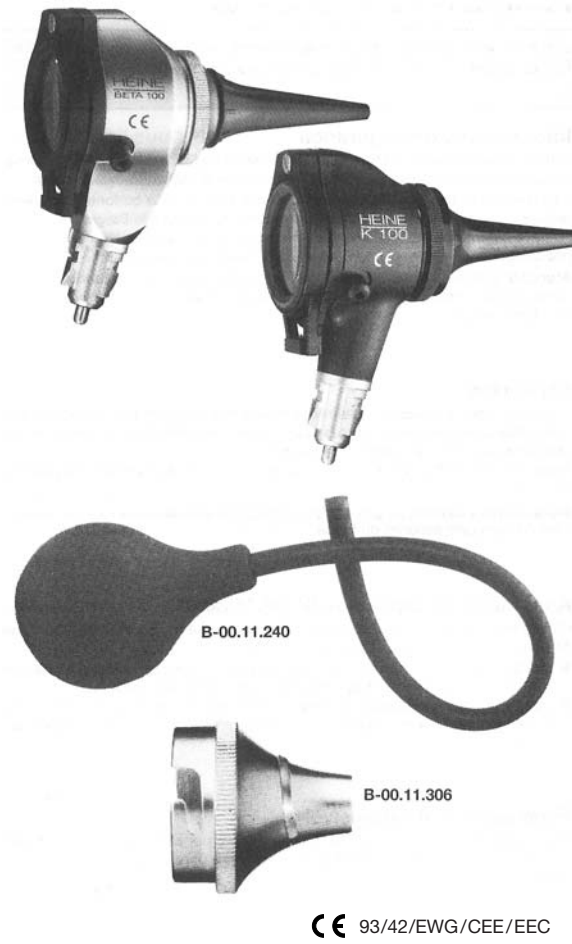HEINE UniSpec® Tip-Spender

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Instructions

HEINE BETA100® and HEINE K100® Diagnostic-Otoscopies

Before using the HEINE Diagnostic-Otoscope read these instructions carefully and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information
Range of applications: The HEINE Diagnostic-Otoscope is used for the examination of the external ear and for general non-invasive examination, including nasal examination.
Safety in use: Use only handle or power sources which are specifically designed for medical use should be used. The HEINE Diagnostic-Otoscope should only be inserted into the ear when a tip or speculum is fitted. Pressure should be applied with great care when doing pneumatic testing.
Warning: The spreadable nasal speculum B-00.11.231 must be used with care in order to avoid damage to the nostril which could be caused by excessive opening of the jaws.

Specula

Reusable ear-specula and nasal specula are attached to the Otoscope so that the peg engages in the bayonet slit. A twist to the right ensures that they are firmly located.

Disposable ear specula can be push-fitted to the otoscope.
General illumination: The HEINE Diagnostic-Otoscope can be used with or without speculum or tip for general illumination of e.g. the oral cavity or skin. Avoid contact with sensitive or damaged skin.

Tip-adaptor (B-00.11.306)

By means of the tip-adaptor, **AllSpec disposable tips** can be used with the HEINE Diagnostic-Otoscope:

- Fit the adaptor to the otoscope so that the pin locates in the bayonet connector. Rotate clockwise to lock in position.
- Fit the AllSpec tip to the adaptor, so that the internal cam locates in the slit in the body of the adaptor and secure by gently rotating the tip clockwise.

Spreadable nasal speculum

This speculum is held in place in the instrument head by a double-sided metal clamp. This can be adjusted by bending it carefully to ensure a firm fit in the head.

Magnification Optics

The swivel lens which is attached to the otoscope gives approx. 3,5 x magnification. In the closed position, pneumatic tests can be made by means of insufflator B-00.11.240.

Changing the bulb

Please note: The performance of this instrument can only be guaranteed if genuine **HEINE bulbs** are used.

- Remove the specula and pull the shield (B-00.11.212) off the bulb.
- Unscrew the bulb.
- Ensure that a bulb of the correct voltage with regard to your handle or transformer is screwed into place.
- Replace the shield.** This reduces stray light and reflections from the inside of the specula and should always be fitted.

Sterilization and Cleaning

Instruments can be gas-sterilized, although occasional cleaning with non-aggressive cleaning agents is usually sufficient. Plastic specula made of SANALON® S and nasal specula can be cleaned and disinfected with the usual solutions and can be boiled. They can also be sterilized as often as desired up to 140 °C.

AllSpec tips and disposable specula are for single use only. Do not attempt to clean them as the tip/specula may be damaged and could injure the patient.

Accessories and replacement parts	
HEINE XHL® Xenon Halogen bulb 2,5 V	X-01.88.037
HEINE XHL® Xenon Halogen bulb 3,5 V	X-02.88.049
Shield for otoscope bulb	B-00.11.212
Insufflator	B-00.11.240
SANALON® S specula (reusable)	
1 complete set = 6 pcs.	B-00.11.221
Spreadable nasal speculum	B-00.11.231
Symmetrically-spreadable universal speculum	B-00.11.239
UniSpec disposable ear-specula:	
1 box = 1000 pcs., 2,5 mm distal dia.	B-00.11.242
idem 4,0 mm distal dia.	B-00.11.241
UniSpec Dispenser	B-00.11.146
Tip-Adapter (for AllSpec disposable tips for use with diagnostic otoscope)	B-00.11.306
AllSpec disposable tips:	
1 box = 1000 pcs., 2,5 mm distal dia.	B-00.11.128
same 4,0 mm distal dia.	B-00.11.127
AllSpec Tip-Dispenser	B-00.11.145

Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

<hr/>	<hr/>
Bought from	Acheté chez
<hr/>	<hr/>
date	le
<hr/>	<hr/>

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