DE **Nacken-Hals-Heizkissen HKH**

GB **Neck Heating Pad HKH**

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**Thank you
very much**

Thank you for your confidence in us and congratulations!

You have purchased the Neck heating pad **HKH**, a quality product by **MEDISANA**.

To ensure the best results and long-term satisfaction with your **MEDISANA** Neck heating pad **HKH**, we recommend that you read the following operating and maintenance instructions carefully.

IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use.

Should you give this device to another person, it is vital that you also pass on these instructions for use.

You will find the following symbols in the operating manual:

This instruction manual belongs to this device. It contains important information about starting up and operation.

Read the instruction manual thoroughly.

Non-observance of these instructions can result in serious injury or damage to the device.

WARNING

These warning notes must be observed to prevent any injury to the user.

CAUTION

These notes must be observed to prevent any damage to the device.

NOTE

These notes give you useful additional information on the installation or operation.

Do not puncture the heated cushion.

Do not use the heated cushion when it is folded.

Only use the heated cushion indoors.

Do not chlorinate!

Do not iron the heated cushion.
You may iron the cushion cover at a low temperature.

Do not dry clean!

Do not dry the heated cushion in a tumble drier.
You may dry the cushion cover in a tumble drier.

Do not wash the heated cushion.
You may wash the cushion cover in a normal wash at max. 40 °C.

1.1 Safety Information

- Carefully check the heat pad before each use, if you notice wear or damage.
- Do **not** use if you notice wear, damage or signs of improper use on the heat pad, the switch or the cables. Return it to the supplier.
- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Do not fold or crease the heat pad during use.
- Neither pierce the pad with, nor attach safety pins or other spiky or sharp objects to the pad.
- Do not use the heat pad on children or persons who are disabled, asleep or insensitive to heat.
- Children should be supervised to ensure that they do not play with the appliance.
- Never use the pad unsupervised.
- The unit is designed for private home use only.
- Do not fall asleep while the heat pad is switched on.
- Prolonged use of the heated cushion at a high setting could lead to skin burns.
- Do not cover the pad with any other cushions. The control switch should not be covered or placed on or below the pad while the unit is in operation.
- Do not sit on the heating pad, only place the pad on the part of your neck you want to treat.
- Never touch a heat pad that has fallen into water. Unplug the unit from the mains outlet immediately.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or turn the heat pad by the mains lead and do not let the lead get tangled.
- Do not use the heat pad when wet and only use it in dry environments (not in the bathroom or similar environments).
- The controls and leads must not be exposed to any kind of moisture.
- If a fault occurs, do not attempt to repair the unit yourself. Repairs must only be carried out by an authorized specialist dealer or other suitably qualified personnel.
- A damaged power supply cable should not be replaced. If the power supply cable is damaged, the whole heating pad must be disposed of.
- When storing the heat pad, allow it to cool down before folding.

1.2 Health tips and recommendations

- Should you have any concerns about health issues, consult your doctor before using the heat pad.
- Do not use the heat pad on parts of the body that are swollen, injured or inflamed.
- Should you experience persistent pains in the muscles and joints, please inform your doctor. Persistent pains can be a symptom of serious illness.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

2.1 Items supplied and packaging

Please check first of all that the unit is complete and is not damaged in any way. In in doubt, do not use the appliance. Send it to a service point.

The following parts are included as standard:

- 1 **MEDISANA** neck heating pad **HKH** with cover
- 1 Instructions for use

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.

If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

Please ensure that polybag packing is kept away from reach of children! Risk of suffocation!

2.2 Use

The **MEDISANA** heating pad **HKH** allows you to gently heat selected parts of the neck. The use of heat promotes blood circulation and boosts the metabolism, thereby increasing the supply of oxygen and nutrients to the muscle tissue. This improves movement in the neck and gives you a pleasant relaxed feeling after a long hard day.

The heat pad features an electric temperature control that regulates the temperature according to the chosen setting.

Only use the heat pad in the cover supplied with it.

Place the heating pad around your neck with the opening at the front and close the fastener.

2.3 Operation

Plug the unit into the mains outlet and move the slider control from position **0** to position **1**. The power LED will light up in red. After a few minutes you will be able to feel the pad getting noticeably warmer. If you require a higher temperature setting, move the switch to the next positions, **2** or **3**, or to the maximum heat setting, position **4**. If the pad gets too hot for you, move the slider back to position **3**, **2** or **1**.

After a short while the heating pad cools down noticeably.

To switch the unit off, move the switch back to position **0**. The red power LED will go out, indicating that the unit is switched off. Unplug the unit from the mains outlet.

Move the selector switch to position **1** or **2** before falling asleep or any continuous use. The heat pad will switch off automatically after approx. 90 minutes of continuous use. To switch it back on again, move the slider control to position **0** and then back to the desired heat setting.

Unplug the unit from the mains outlet once you have finished using it.

**3.1
Cleaning and maintenance**

Heat pad

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heat pad can be cleaned with a soft brush when dry.
- Never use strong detergents or hard brushes.
- You should untangle the cable from time to time.
- When storing the appliance, allow it to cool down. Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

Heat pad cover

- Remove the heat pad from its cover.
- Dry clean the removable cover or as detailed in the instructions on the label.

**3.2
Disposal**

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

**3.3
Specifications**

Name und Modell	: MEDISANA neck heating pad HKH
Power Supply	: 220-240 V AC 50 Hz
Heat output	: approx. 45 watts
Automatic switch off	: after approx. 90 min
Switching levels	: 0 - 1 - 2 - 3 - 4
Protection class II	:
Operating conditions	: Only use in dry rooms as described in the operating instructions
Storage conditions	: Laid out flat, clean and dry
Dimensions	: approx. 44,5 x 28 cm
Weight	: approx. 410 g
Mains lead length	: approx. 2,75
Article No.	: 60151
EAN Code	: 40 15588 60151 4

In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.

4.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9

40724 Hilden

Germany

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: info@medisana.de

Internet: www.medisana.de

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB