HEINE C 2.3[®]-Binokularlupe

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0)81 52/38-0 · Fax +49 (0)81 52/38-202
E-mail: info@heineopto.de · www.heine.com
med 2199 1/6.06

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HEINE C 2.3[®] Binokularlupe

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Instructions for use

HEINE C 2.3[®] Binocular Loupe

Before using this HEINE C 2.3[®] binocular loupe, read these instructions carefully and keep them in a safe place for future reference.

Safety informations

Range of applications: Use only as a magnifier for examination and operations.

Safety in use: Do not look into the sun or into a powerful light source with this loupe.

Adjustment

To get the best results from this loupe, the optics must be adjusted exactly. This is done as follows:

Getting a single clear image

- Unscrew both set screws so that the eyepieces can be slid in or out.
- Put on the loupe and look at a well lit three-dimensional object at working distance.
- Adjust one eyepiece after the other until you see a round image with no shadows and a focused object in each.
- Screw in both set screws softly without moving the eyepieces.
- Finally screw in both set screws firmly.

Spectacle Frame

The frame can be adapted to your needs by gently bending the nasal supports and adjusting the temple length to ensure that it is comfortable.

Care and Maintenance

Treat the HEINE C 2.3[®] binocular loupe like any other delicate optical instrument and store it in a dust-free place or in the case provided. Clean the glass surfaces with a micro-fiber cloth.

All other surfaces and accessories can be wiped clean with a damp cloth with a little alcohol or non-aggressive cleaner. The optional lens protectors can be cleaned at temperatures of up to 100 °C (212 °F) with a non-aggressive cleaner.

Technical Specification

Magnification*:	2.3x
Working distance*:	340 mm and 450 mm
Pupillary distance:	50 – 74 mm
Weight:	46 g
Frame:	Metal, rhodium-plated

*: For non-corrected refractive errors working distance will differ from the above values.

As a general rule the working distance will decrease with near-sightedness and increase with far-sightedness.

Warranty

We guarantee this device for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts. Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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